
ENGLISH

Before first-time use of the ProBP 3400, complete the following steps:

Insert the battery

The battery is shipped separately from the ProBP 3400 device. You must install and charge the battery for six hours before the device can be used.

- Remove the screw from the battery compartment door with a Phillips screwdriver.
- Remove the battery compartment door.
- Remove the battery from the shipping bag.
- Using the polarized guidelines on the battery label, insert the battery, positive (+) side first, into the battery compartment.
- Replace the battery compartment door.
- Secure the screw with the screwdriver.

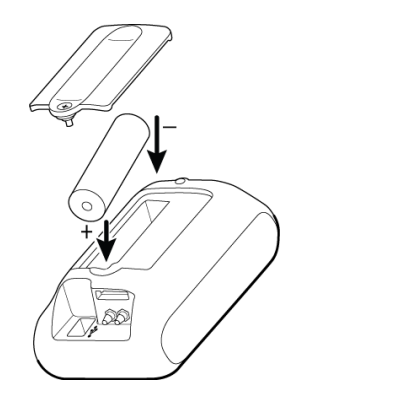Mount the device

For mounting instructions, see the accessory's directions for use. For mounting options, see the ProBP 3400 approved accessories in the Appendix.

Assemble power cord and wall plug/line cord

- Align the grooves and two prong plug in the wall plug or line cord with the casing and plug receptacle located on the transformer.
- Slide the wall plug or line cord onto the transformer until it is firmly seated.

Charge the ProBP 3400

- To charge the device:
- Insert the USB B connector into the USB/ external power connection port on the back of the device.
 - Insert the other end of the USB cord into USB connector on the power transformer, then plug the transformer into the main power source.

Initial startup

To complete initial configuration, you must turn on the device, and select an operating language, date, and time.

- Press the **Power on/power off** button. After device power up, the display window shows the Language screen.

- Use the Up and Down navigation buttons to highlight the correct option.

- Press the **Select** button to choose a language. A confirmation window displays your language selection.

Set date and date format

After you select a language, set the device Date and Date format:

- The Date format field is highlighted. Press the **Up navigation** or **Down navigation** buttons to access the correct format option.
- Press the **Select** button to accept the date format as shown.
- Press the **Down navigation** button to highlight the date.
- Press the **Select** button. The first numeric field is highlighted for editing.
- Use the **Up navigation** button to increase the number value; use the **Down navigation** button to decrease the number value.
- Press the **Right navigation** button to move to the next field.
- Repeat steps 5 and 6 to edit additional numeric values.
- Press the **Right navigation** button to highlight the **OK** button.
- Press the **Select** button to accept the date as shown.
- Press the **Down navigation** button to highlight the **Next** button.
- Press the **Select** button to move to the Time and Time format screen.

Set time and time format

- The Time format field is highlighted. Press the **Up navigation** or **Down navigation** buttons to access the correct format option.
- Press the **Select** button to accept the time format as shown.
- Press the **Down navigation** button to highlight the time.
- Press the **Select** button. The first numeric field is highlighted for editing.
- Use the **Up navigation** button to increase the number value; use the **Down navigation** button to decrease the number value.
- Press the **Right navigation** button to move to the next field.
- Repeat steps 5 and 6 to edit additional numeric values.
- Press the **Right navigation** button to highlight the **OK** button.
- Press the **Select** button to accept the entry.
- Press the **Down navigation** button to highlight the **Next** button.
- Press the **Select** button to complete the Setup process. The device will display the Home screen.

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